
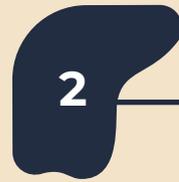


KORA STRIP

Testing Instructions

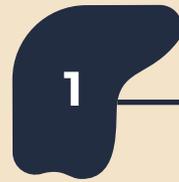


Storage Conditions: ❄️


Keep your Kora Strip Kit refrigerated until use. We recommend keeping the whole box in your fridge. If needed, carefully place the Kora Strips in the fridge, and keep the rest of the kit safe and handy for when you are ready to test.

Step 1 - Prepare Your Test Kit


Your Kora Strip Kit comes with:

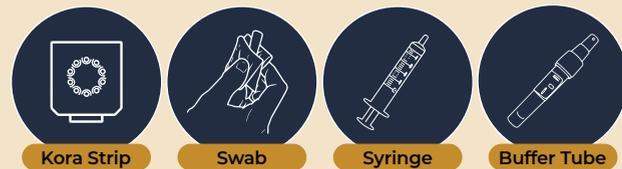

1. Remove the sticker protecting the sensors on the Kora Strip, and shake off any liquid in the sensor wells. Be careful not to touch the sensors.
2. Turn on the BioReader, then insert the Kora Strip into the BioReader, with the sensors facing up.
3. Pair your phone to the BioReader using your PRONOVA app by starting a test.

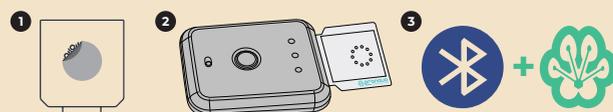

Step 2 - Collect Your Sample

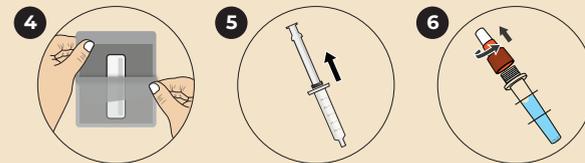

4. Open the swab, being careful not to drop it. Hold it by one end with two fingers.
5. Open the syringe and pull out the inner stick.
6. Remove the colored cap off the buffer tube, being careful not to spill the liquid inside.

7. While holding the swab by one end with two fingers, gently rub it around the inside of the cheeks, focusing on the back areas where the saliva pools. Rub for 30 seconds on each side of the mouth. It is important that the swab absorbs enough saliva.




If necessary, gently hold your dog's snout while swabbing to help with the process.




For more information, go to www.pronovapets.com

Step 3 - Process Your Sample

8. Insert the wet swab into the open syringe, with the wet end going in first.
9. Re-insert the syringe stick, being careful to not compress the swab yet.
10. Then, squeeze the syringe so it compresses the swab, squeezing out the saliva into the opened buffer tube. Fill the buffer tube as close to the second line as possible. In the PRONOVA app, you will be able to mark what level of saliva you collected once you start the test on the app.

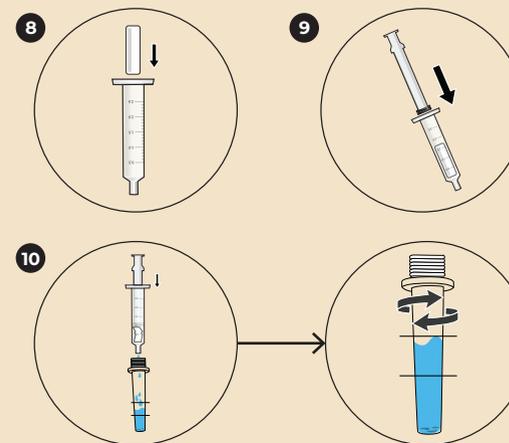

11. Once you have collected enough saliva, put the colored cap back on the tube. Then, shake gently for 5 seconds to mix the sample.

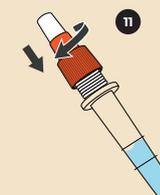

Step 4 - Add Sample



12. Remove the small white cap on the buffer tube, and carefully squeeze 2 drops into each sensor well on the Kora Strip. Make sure the entire sensor is covered with the sample. Gently tap the Kora Strip to help the liquid spread if necessary. When you are ready, start the test by pressing start on the app or the button on the BioReader. The test will take 15 minutes.



Step 5: Interpreting Results

The app will read the data from the BioReader for the last test completed. Once the data is processed, you will be able to see the results in your dog's profile, along with actionable health insights for each biomarker.

Dispose of the test kit: Once the test is completed, dispose of the used swab, syringe, buffer tube and Kora strip.

